



**HBV Longitudinal Panel
Part Number: HBV6512**

*These reagents are not a substitute for the mandatory positive and negative control reagents provided with licensed test kits.
For Research Use Only. Not for use in diagnostic procedures.*

PANEL NUMBER	DATE OF DRAW	ORTHO VITROS ECi ANTI-HIV ½	ORTHO VITROS ECi ANTI-HCV	ABBOTT HBsAg	ORTHO VITROS ECi HBsAg	ABBOTT ANTI-HBc TOTAL	ORTHO VITROS ECi ANTI-HBc TOTAL	ABBOTT ANTI-HBc IgM
		TEST DATE: 8/8/2014	TEST DATE: 8/8/2014	TEST DATE: UNK	TEST DATE: 8/8/2014	TEST DATE: UNK	TEST DATE: 8/8/2014	TEST DATE: UNK
6512-01	05/30/01	0.08	0.19	>40.8	9870	0.07	0.03	>6.9
6512-02	06/13/01	0.08	0.16	>40.8	2220	0.08	0.06	>6.9
6512-03	06/27/01	0.09	0.11	18.8	44.9	0.09	0.03	>6.9
6512-04	07/11/01	0.08	0.12	1.6	2.41	0.11	0.04	>6.9
6512-05	07/25/01	0.06	0.13	0.3	0.06	0.03	0.03	>6.5
6512-06	08/08/01	0.09	0.07	0.1	0.05	0.03	0.02	5.9
6512-07	08/22/01	0.09	0.13	0.1	0.07	0.04	0.02	5.9
6512-08	09/05/01	0.09	0.11	0.1	0.06	0.04	0.03	5.1
6512-09	09/19/01	0.08	0.10	0.8	0.05	0.03	0.02	4.7
6512-10	10/03/01	0.08	0.27	0.02	0.06	0.03	0.03	4.1
6512-11	10/17/01	0.09	0.07	0.1	0.05	0.05	0.02	5.8
6512-12	10/30/01	0.09	0.10	0.02	0.05	0.04	0.02	5.4
6512-13	12/11/01	0.07	0.04	0.1	0.06	0.06	0.02	4.4
6512-14	12/25/01	0.09	0.07	0.1	0.09	0.06	0.03	4.1
6512-15	01/08/02	0.10	0.09	0.1	0.05	0.05	0.02	4.1

PANEL NUMBER	ORTHO VITROS ECi ANTI-HBc IgM	DIASORIN ANTI-HBe	ORTHO VITROS ECi ANTI-HBe	ABBOTT ANTI-HBs QUANT	ORTHO VITROS ECi ANTI-HBs QUANT	ORTHO VITROS ECi HBeAg	DIASORIN HBeAg
	TEST DATE: 8/8/2014	TEST DATE: UNK	TEST DATE: 8/8/2014	TEST DATE: UNK	TEST DATE: 8/8/2014	TEST DATE: 8/8/2014	TEST DATE: UNK
6512-01	3.80	0.6	0.53	0.3	1.5	0.72	0.5
6512-02	3.83	0.9	0.27	0.5	1.6	1.26	0.4
6512-03	3.57	1.0	0.38	0.3	1.5	0.20	0.2
6512-04	3.34	1.0	0.50	0.3	1.2	0.13	0.2
6512-05	3.08	0.3	0.85	0.4	1.1	0.12	0.3
6512-06	2.65	0.4	0.90	0.5	2.8	0.13	0.2
6512-07	2.73	0.3	1.71	0.6	2.5	0.13	0.3
6512-08	2.60	0.2	1.73	0.7	3.1	0.13	0.2
6512-09	2.48	0.2	1.73	0.9	3.7	0.12	0.3
6512-10	2.31	0.2	1.45	1.1	5.9	0.14	0.2
6512-11	2.51	0.2	2.70	1.1	7.1	0.12	0.3
6512-12	2.40	0.2	1.83	1.2	9.7	0.13	0.2
6512-13	2.20	0.2	1.27	1.6	8.8	0.15	0.4
6512-14	QNS	0.3	1.03	1.5	QNS	0.13	0.2
6512-15	2.11	0.4	0.78	1.31	8.0	0.12	0.3

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV6512 Rev02
Effective Date: 09/01/2021