



**HBV Longitudinal Panel  
Part Number: HBV6537**

*These reagents are not a substitute for the mandatory positive and negative control reagents provided with licensed test kits.  
For Research Use Only. Not for use in diagnostic procedures.*

PANEL NUMBER	DATE OF DRAW	ORTHO VITROS ECi ANTI-HIV ½	ABBOTT HBsAg	ORTHO VITROS ECi HBsAg	ABBOTT ANTI-HBc TOTAL	ORTHO VITROS ECi ANTI-HBc TOTAL	ABBOTT ANTI-HBc IgM
		TEST DATE: 8/21/2014	TEST DATE: UNK	TEST DATE: 8/21/2014	TEST DATE: UNK	TEST DATE: 8/21/2014	TEST DATE: UNK
6537-01	11/28/01	0.09	>37	4420	0.03	0.02	>6.5
6537-02	12/12/01	0.11	>37	6520	0.1	0.03	>6.5
6537-03	12/26/01	0.11	>37	9990	0.1	0.07	>6.5
6537-04	01/08/02	0.10	>37	3140	0.1	0.12	>6.5
6537-05	01/27/02	0.12	>37	1410	0.04	0.02	1.9
6537-06	02/05/02	0.10	1.1	0.35	0.1	0.14	4.0
6537-07	02/19/02	0.10	0.3	0.09	0.1	0.16	2.9
6537-08	03/05/02	0.12	0.3	0.23	0.1	0.15	2.5
6537-09	03/19/02	0.12	0.1	0.08	0.1	0.15	2.5
6537-10	04/02/02	0.13	0.1	0.11	0.1	0.14	2.4
6537-11	04/16/02	0.12	0.1	0.13	0.1	0.13	2.2
6537-12	04/30/02	0.12	0.1	0.17	0.1	0.13	2.2

PANEL NUMBER	ORTHO VITROS ECi ANTI-HBc IgM	DIASORIN ANTI-HBe	ORTHO VITROS ECi ANTI-HBe	ABBOTT ANTI-HBs QUANT	ORTHO VITROS ECi ANTI-HBs QUANT	ORTHO VITROS ECi HBeAg	DIASORIN HBeAg
	TEST DATE: 8/21/2014	TEST DATE: UNK	TEST DATE: 8/21/2014	TEST DATE: UNK	TEST DATE: 8/21/2014	TEST DATE: 8/21/2014	TEST DATE: UNK
6537-01	3.68	0.1	2.72	0.2	1.7	0.15	0.05
6537-02	3.58	0.06	18.3	0.5	0.6	0.20	0.09
6537-03	3.11	0.05	17.3	0.3	0.9	0.20	0.04
6537-04	2.64	0.05	19.2	0.3	1.5	0.18	0.19
6537-05	1.01	0.06	29.8	0.3	1.8	0.16	0.08
6537-06	1.74	0.06	22.1	0.3	2.4	0.18	0.14
6537-07	1.29	0.05	25.1	0.5	1.6	0.15	0.07
6537-08	1.27	0.05	12.6	0.6	2.5	0.36	0.15
6537-09	1.12	0.04	24.7	0.7	5.8	0.15	0.01
6537-10	1.09	0.05	28.2	1.8	14.1	0.15	0.11
6537-11	1.03	0.05	17.3	3.4	33.5	0.24	0.04
6537-12	0.90	0.05	25.8	3.1	43.4	0.15	0.11

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHBV6537 Rev02  
Effective Date: 09/07/2021