



**HBV Longitudinal Panel
Part Number: HBV6546**

*These reagents are not a substitute for the mandatory positive and negative control reagents provided with licensed test kits.
For Research Use Only. Not for use in diagnostic procedures .*

***Donor is co-infected with Hepatitis C.**

| PANEL NUMBER | DATE OF DRAW | ORTHO VITROS ECI ANTI-HIV ½ | ORTHO VITROS ECI ANTI-HCV | ABBOTT HBsAg | ORTHO VITROS ECI HBsAg | ABBOTT ANTI-HBc TOTAL | ORTHO VITROS ECI ANTI-HBc TOTAL | ABBOTT ANTI-HBc IgM |
|--------------|--------------|-----------------------------|---------------------------|----------------|------------------------|-----------------------|---------------------------------|---------------------|
| | | TEST DATE: 8/21/2014 | TEST DATE: 8/21/2014 | TEST DATE: UNK | TEST DATE: 8/21/2014 | TEST DATE: UNK | TEST DATE: 8/21/2014 | TEST DATE: UNK |
| 6546-01 | 01/28/02 | 0.11 | 20.9 | >36.4 | 7790 | 0.1 | 0.15 | >5.5 |
| 6546-02 | 02/11/02 | 0.12 | 22.3 | >36.4 | 7490 | 0.1 | 0.03 | >5.5 |
| 6546-03 | 02/25/02 | 0.1 | 24.9 | >36.4 | 2090 | 0.1 | 0.03 | 4.1 |
| 6546-04 | 03/11/02 | 0.11 | 27.7 | >36.4 | 177 | 0.1 | 0.03 | 2.7 |
| 6546-05 | 03/25/02 | 0.10 | 28.5 | 1.9 | 0.27 | 0.1 | 0.03 | 1.9 |
| 6546-06 | 04/08/02 | 0.17 | 27.3 | 0.3 | 0.07 | 0.1 | 0.03 | 1.4 |
| 6546-07 | 04/22/02 | 0.14 | 26.2 | 0.2 | 0.06 | 0.1 | 0.04 | 1.04 |
| 6546-08 | 05/06/02 | 0.13 | 24.9 | 0.1 | 0.05 | 0.1 | 0.02 | 1.2 |
| 6546-09 | 05/20/02 | 0.12 | 23.3 | 0.04 | 0.08 | 0.1 | 0.03 | 0.8 |
| 6546-10 | 06/03/02 | 0.12 | 23.5 | 0.07 | 0.08 | 0.1 | 0.02 | 0.9 |
| 6546-11 | 06/17/02 | 0.13 | 21.9 | 0.02 | 0.07 | 0.1 | 0.04 | 0.6 |
| 6546-12 | 07/01/02 | 0.12 | 20.7 | 0.04 | 0.06 | 0.1 | 0.02 | 0.5 |

| PANEL NUMBER | ORTHO VITROS ECI ANTI-HBc IgM | DIASORIN ANTI-HBe | ORTHO VITROS ECI ANTI-HBe | ABBOTT ANTI-HBs QUANT | ORTHO VITROS ECI ANTI-HBs QUANT | ORTHO VITROS ECI HBeAg | DIASORIN HBeAg |
|--------------|-------------------------------|-------------------|---------------------------|-----------------------|---------------------------------|------------------------|----------------|
| | TEST DATE: 8/21/2014 | TEST DATE: UNK | TEST DATE: 8/21/2014 | TEST DATE: UNK | TEST DATE: 8/21/2014 | TEST DATE: 8/21/2014 | TEST DATE: UNK |
| 6546-01 | 3.64 | 0.2 | 0.90 | 0.3 | 1.6 | 0.15 | 0.07 |
| 6546-02 | 3.19 | 0.09 | 3.86 | 0.4 | 2.1 | 0.14 | 0.10 |
| 6546-03 | 2.23 | 0.05 | 5.86 | 0.3 | 1.9 | 0.14 | 0.04 |
| 6546-04 | 1.56 | 0.05 | QNS | 0.3 | 2.7 | 0.13 | 0.15 |
| 6546-05 | 0.98 | 0.04 | 19.7 | 0.5 | 3.1 | 0.13 | 0.07 |
| 6546-06 | 0.77 | 0.05 | 25.7 | 0.6 | 1.8 | 0.12 | 0.12 |
| 6546-07 | 0.56 | 0.02 | 28.1 | 0.8 | 3.0 | 0.12 | 0.08 |
| 6546-08 | 0.49 | 0.03 | 30.4 | 1.12 | 6.3 | 0.13 | 0.11 |
| 6546-09 | 0.37 | 0.01 | 27.8 | 1.11 | 7.6 | 0.16 | 0.01 |
| 6546-10 | 0.37 | 0.01 | 30.6 | 1.5 | 11.5 | 0.15 | 0.11 |
| 6546-11 | 0.28 | 0.01 | 25.8 | 1.4 | 12.0 | 0.17 | 0.05 |
| 6546-12 | 0.28 | 0.02 | 28.3 | 1.0 | 13.6 | 0.14 | 0.12 |

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV6546 Rev02
Effective Date: 09/22/2021