Orthopaedic Line for  
Children

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# Baby System<sup>®</sup>

DEVELOPMENTAL DYSPLASIA  
AND LUXATION OF THE HIP (DDH)  
Graf's method

Type IIa(-)

Type IIa(-) and IIb

Type IIc

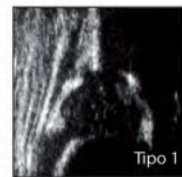
Type IIa(-), IIb and IIc

Graf's Cradle

# PREVENTION PROGRAMME


## DEVELOPMENTAL DYSPLASIA AND LUXATION OF THE HIP

Type 1

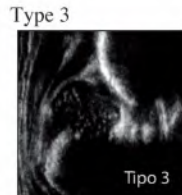Type 2

Type 3

Type 4

Hip	Classification according to Graf	Bony Acetabulum Alpha Bony Angle	Acetabulum Rim	Cartilaginous Acetabulum Beta Cartilaginous Angle	Age	Note	Therapy
	<b>TYPE I</b> normal hip	good $\alpha \geq 60^\circ$	Ia angular Ib smoothed	covering Ia $\beta < 55^\circ$ (thin, well-covering) Ia $\beta > 55^\circ$ (large base, thin covering)	any age		Prevention (from birth to ultrasound scanning): Baby System Prävention
	<b>TYPE IIa (+)</b> developmentally immatur, adequate to age	sufficient with respect to age $\alpha = 50-59^\circ$ (sufficiently well-developed – see scanmeter)	rounded	covering	0-12 weeks		Baby System Speizhose
	<b>TYPE IIa (-)</b> developmentally immature, maturity deficiency	abnormal $\alpha = 50-59^\circ$ (insufficiently well-developed – see scanmeter)	rounded	covering	> 6-12 weeks		Baby System Dynamische Speizhose
	<b>TYPE IIb</b> late ossification	abnormal $\alpha = 50-59^\circ$	rounded	covering	> 12 weeks		
	<b>Exception: TYPE II</b> in case of late ossification	abnormal	angular	covering (exogenous because of late ossification)	any age		
	<b>TYPE IIc</b> (decentring hip) IIc stable/IIc unstable	severely abnormal $\alpha = 43-49^\circ$	rounded/flat	slightly covering $\beta < 77^\circ$	any age		Baby System Korrektur Latzhose orthopaedic examination
	<b>TYPE D</b> hip about to decentring $\beta > 77^\circ$	severely abnormal $\alpha = 43-49^\circ$	rounded/flat	compressed $\beta > 77^\circ$	any age		orthopaedic examination
	<b>TYPE IIIa</b> decentred hip $\alpha < 43^\circ$	insufficient $\alpha < 43^\circ$	flat	cranially compressed, without alteration of the cartilaginous roof (absence of echogenicity)	any age		Speizhose Superior Graf
	<b>TYPE IIIb</b> decentred hip $\alpha < 43^\circ$	insufficient $\alpha < 43^\circ$	flat	cranially compressed, without alteration of the cartilaginous roof (absence of echogenicity)	any age		
	<b>TYPE IV</b> decentred hip $\alpha < 43^\circ$	insufficient $\alpha < 43^\circ$	flat	caudally compressed, proximal perichondrium with horizontal or lowering course	any age		

Baby System®

Prevention

Type IIa(-)

Type IIa(-) and IIb

Type IIc

Type IIa(-), IIb and IIc

# Baby System®

Orthopaedic Line for  
Children

## PREVENTION OF THE DEVELOPMENTAL DYSPLASIA OF THE HIP

The Baby System Soft Nappy is the most effective alternative to the so-called "double nappy". It's an ideal preventive appliance and if used in the early days of life it will help the physiological posture of the newborn's lower limbs, as it stimulates the normal morphological hip development.

**One size only** - Art.202: it suits babies from birth to 6-7kg

**Smaller size** - Art.201: for premature babies only

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Orthopaedic Line for  
Children

## BABY SYSTEM CORRECTIVE NAPPY

The Baby System Corrective Nappy promotes a prompt and easy separation of a baby's limbs while they are still developing. It also encourages correct maturation at the hip joint even when a late ossification is detected.

### Available in 3 sizes:

**Size 1** - Art.301 (semi-rigid only): from birth to 5-6kg (approximately a 14cm abduction)

**Size 2** - Art.302 (rigid only): for babies from 6 to 9 kg (approximately a 16cm abduction)

**Size 3** - Art.303 (rigid only): for children from 9 to 15kg (approximately a 18cm abduction)

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Orthopaedic Line for  
Children

## BABY SYSTEM CORRECTIVE SALOPETTE

The Baby System Corrective Salopette is the most effective and safe alternative to traditional therapies in treating hip dysplasia and dislocation during the first months of life of a newborn child.

### Available in 4 sizes:

**Size 0** - Art.400: Approximately for premature babies (14 or 16cm abduction)

**Size 1** - Art.401: Approximate abduction: 18 or 20cm

**Size 2** - Art.402: Approximate abduction: 20 or 22cm

**Size 3** - Art.403: Approximate abduction: 22 or 24cm

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# Baby System®

## Rapid-Eco & Eco-Support

... precision and safety as developed and devised by Prof. Graf

### Rapid-Eco (Art. 5000)

#### Ultrasound Positioning Device – Graf Method

Rapid-Eco is a piece of equipment which is designed to hold and position a newborn baby quickly and efficiently for a scan. On this piece of equipment the newborn is kept firmly still and in the correct position without needing the assistance of a third person, this is because the pressure from both sides of the equipment blocks the movement of the baby with a pressure which is in proportion to the baby's weight and hence the doctor is able to carry on the medical examination in a quick and precise way. Rapid-Eco is washable and made from materials of advanced technology. The Rapid-Eco gives doctors an easier and quicker way of working. (Measurements in cm: 75 length x 42 depth x 20 height)

### Eco-Support (Art. 5001)

#### Arm guide for ultrasound probes – Graf Method

This support for ultrasound probes is made of steel and is fixed to the base of the Rapid-Eco. During the scan, the doctor is able to observe and analyse the data on the monitor, as the Eco-Support allows precise movement and hence avoids errors in the diagnosis. (Measurements in cm: 48 height)

CHECK PRICE Online

## DESIGNED, CERTIFIED AND AUTHORIZED BY PROF. GRAF

*I subscribed Prof. Reinhard Graf, I authorize the 'Povatex' company of Preite Vita s.a.s. to use and put my name on the positioning devices for the hip ultrasound produced by it "Eco-Support And Rapid-Eco" and bring it up in all countries where 'Povatex s.a.s.' will be present to sell the above-mentioned ecographic supports.*

Signature  
PROF. REINHARD GRAF

Manufacturer - <https://www.povatex.it>  
**POVATEX** s.a.s.  
CE patented products MADE IN ITALY

**H-H-C** e-mail : [incoming@h-h-c.com](mailto:incoming@h-h-c.com)  
OFFICIAL  
DISTRIBUTOR  
<https://www.h-h-c.com>  
LRQA CERTIFIED ISO 13485  
UNAS